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- 1. Product Name :** Optimos™ Polypectomy Snare
- 2. Description**  
The Optimos™ Polypectomy Snare is designed to be connected to an ESU (electrosurgical unit) for removing polyps. The tissue to be removed can be excised by placing a finger on the handle and tightening the loop. Loops of appropriate sizes and shapes can be selected and used based on the locations and sizes of the polyps. It has a shaft that transmits force to the loop so that the loop can snare polyps, and a sheath that protects the shaft. The handle transmits force to the sheath to tighten or loosen the snare, and the rotation of the loop can be controlled using the locking controller located on the handle.
- 3. Indications for Use**  
To excise tissues in the body such as adenomas and polyps
- 4. Contraindications**  
Not to be used for purposes other than those specifically described in Indications for Use
- 5. Potential Complications**
  - Bleeding - Perforation - Burns due to overheated ESU - Recurrence of tumors - Hematochezia, diarrhea
  - Pancreatitis - Cholangitis - Papillary stenosis - (Electric) Shock - Stricture
- 6. Precautions**
  - Identify the location and size of the tissue to be excised using an endoscope.
  - Check the state of packaging and sterilization and the expiration date of the product.
  - Make sure you completely understand how to use the product.
  - Before using the product, check that the snare is tightening appropriately by pushing, pulling, and operating the handle several times.
- 7. Cautions**
  - This product is supplied in sterilized packages; do not use this product if its internal package has been damaged due to impacts, etc.
  - If the tissue to be removed is large, remove it by breaking it into several pieces.
  - When this product is inserted using an endoscope, the endoscope should have an accessory channel of an appropriate size to accommodate the product (refer to the attached label).
  - The snare should not be spread until the product is positioned appropriately for the location of the tissue to be removed.
  - If resistance is felt while the product is approaching the tissue to be removed or while the snare is being spread, the use of the product should be stopped.
  - If the wire of the snare is bent or twisted when the snare has been spread, stop using the product.
  - When using the product, do not exceed the maximum limit for output energy specified in the manual of the electrosurgical unit. Excessive energy may cause injury to the patient or damage to the product.
- 8. Warning**
  - Do not touch the loop with your finger or another tool (insulator or components that react to metal) when the product is connected to the ESU.
  - Do not activate the current when the loop is in contact with a metallic object or device, as it may cause unexpected injuries to the patient or damage to the product or other devices.
  - As with all other types of ESUs, do not use this product in places with surgical drapes, inflammable anesthetics, nitrogen oxide, oxygen, or other inflammable materials (the ESU may cause ignition).
  - Do not perform electrosurgery while a combustible anesthetic is being used.
  - Tissue at the end of an activated snare may become an ember, particularly in high-oxygen environments.
  - Before and during each electrosurgery, check that an appropriate amount of energy is output.
  - Take precautions against ignitable gases that are spontaneously generated in the intestine.
  - Use this product only after the field of vision has been secured by using an endoscope, and after completing preparations for surgery.
  - Do not use this product when there is a heart pacemaker within a distance of 4 meters.
  - Avoid using mono-pole electrodes for patients with implantable cardioverter defibrillators.
  - Those who experience allergic reactions to drugs may also show allergic reactions to this operation.
  - This product is disposable.
  - Do not re-sterilize or reuse this product.
  - Do not use this product when the patient has a severe fever, hemorrhaging, or difficulty breathing.
  - Do not use this product if its sterile package is opened or damaged.
  - Use only parts of this product whose shelf life has not expired.
- 9. Instructions for Use**
  - (1) This product should be used by trained doctors only.
  - ⚠️ Follow the user manual of the ESU for attaching the electronic pad.
  - (2) Before using the product, check if the package or product has been damaged.
  - (3) For the smooth operation of the product, push the handle to the end to expand the snare.
  - (4) Check that the cables of the electric equipment are connected to avoid any problem.
  - (5) Connect the cord from the electric equipment to the handle of the product with the electric equipment's power turned off, and check the compatibility between them.
  - ⚠️ Be sure to secure an appropriate environment in accordance with the instructions for surgical environments provided by the equipment manufacturer before starting the equipment.
  - (6) Identify the location of the tissue to be removed using an endoscope.
  - (7) Insert the prepared product through the channel of the endoscope.
  - (8) Push the product forward little by little until the product comes out of the channel of the endoscope.
  - (9) Place the snare on the site of the tissue to be removed. At this time, the direction of the loop may be adjusted by rotating the locking controller of the handle.
  - (10) Turn on the electric equipment and tighten the snare slowly to remove the tissue.  
\*Polyps under 5 mm in diameter can be removed without applying electric energy (cold type model only).
  - (11) After removing the tissue, turn off the equipment, separate the product from the equipment, remove the product from the endoscope, and dispose of the product in accordance with the disposal procedure separately determined by the hospital.
- 10. Handling / Storage**  
The expiration period for this product is three years. Do not use any part of this product if it has expired. Store this product in a dry, clean place at the temperature of 15°C-30°C. Keep the product away from direct sunlight.
- ⚠️ Reuse Precaution Statement**  
This product has been sterilized using ethylene oxide (E.O.) gas. Do not use if the sterile package is opened or damaged. The company is not responsible for re-sterilized products.
- \* Electrosurgical Unit or Generator
  - ERBE VIO 200S

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## TAEWOONG MEDICAL CO., LTD