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1. Description
The Optimos™ Biopsy Forceps are designed so that the user can easily insert them into an accessory channel of the endoscope and manipulate the handle comfortably. The shaft of the forceps is coated with non-toxic stainless steel in a spiral pattern and polyethylene for the patient's safety, with the streamlined cups designed in such a way that they will not damage the endoscope. Moreover, as the cups engage precisely, tissue can be collected safely with minimal bleeding.

2. Indications for Use
The Optimos™ Biopsy Forceps are used for the examination of cellular tissue from the digestive region.

■ Patient Population
Not specialized for a particular population, but applicable to all patients over 18 years of age; however, the decision rests with the physicians

3. Performance and Clinical Benefits
- Performance : Sampling of tissue - Clinical benefit : Histological examination

4. Contraindications
Using the product for purposes other than that specified concretely in the Indications for Use is prohibited. Do not use on patients with hemostatic disorders.

5. Potential Complications
Potential complications related to gastro-endoscopy include, but are not limited to, the following: perforation, bleeding, aspiration, febricity, infection, allergic reaction to drugs, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest, and laceration

6. Precautions

- Secure a suitable size for the endoscope channel for the device (refer to the attached label).
- If possible, keep the endoscope in place when inserting or removing the forceps.
- The outer diameter is 1.8mm or 2.4mm, so an endoscopic channel of over 2.8mm is necessary.
- Before using the product, make sure you have a thorough understanding of the technological principles, such as clinical advantages and disadvantages, recovery from foreign body reactions, biopsies, etc., in relation to gastrointestinal endoscopy. This product must be used only when the relevant biopsy procedure is directed or performed by a doctor who is appropriately qualified for biopsy procedures using an endoscope.
- When inserting the forceps into or removing them from the channel, make sure that the cups are closed. If the cups of the forceps are opened when inserting or removing them, they may cause damage to the endoscope and the forceps themselves.
- When advancing the product through the endoscope channel, do so slowly while keeping the channel and handle in a straight line. Avoid using excessive force, take care not to create any twist when removing the device from the channel.
- When the biopsy site is depressed or over 15 mm, an ER procedure using snares or ESD knives is necessary for precision examination.

△ If any serious incident occurs in relation to this device, the user and/or patient should report to the competent authority of the Member State in which the user and/or patient is established and to the manufacturer.

7. Warnings

- Those who experience allergic reactions to drugs may also show allergic reactions to this operation.
- This product is disposable and should therefore not be reused. If so, risks such as infection may occur.
- Biopsy function can deteriorate if reused, which could pose a risk to the patient.
- Do not re-sterilize or reuse this product.
- Do not use this product when the patient has a severe fever, hemorrhaging, or difficulty in breathing.
- This product is supplied in sterilized packages; do not use this product if its internal package has been damaged due to impact, etc.
- Use only parts of this product whose shelf life has not expired

8. Instructions for Use

- 1) Pre-inspection
 - ① Remove the device from the box.
 - ② Visually inspect the product for any bent, broken, or abnormal part.
 - ③ Gently open and close the cups of the forceps several times to make sure that they close tightly without any problem.
 - ④ If it is impossible to operate the product or any risk factor is found, do not use the product.

- 2) How to insert the product
 - ① Place the endoscope over the patient's lesion.
 - ② By pulling the handle of the product in the direction of the user, pass it through the accessory channel of the endoscope while keeping the cups closed.
 - ③ Move the product to the target tissue to be biopsied, then open the cups and collect the tissue.
 - ④ Slowly remove the product from the accessory channel while keeping the cups engaged by applying appropriate force.
 - ⑤ After the procedure, the doctor can prescribe appropriate medicine for the patient.
 - ⑥ Dispose of the device after the procedure in accordance with the prescribed disposal procedure.

9. Handling / Storage
The expiration date of this product is three years. Do not use any part of this product if it has expired. Store this product in a dry, clean place at a temperature of 15°C-30°C. Keep the product away from direct sunlight.

Reuse Precaution Statement

This product has been sterilized using ethylene oxide (E.O.) gas. Do not use if its sterile package is opened or damaged. The company is not responsible for re-sterilized products.

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TAEWOONG MEDICAL CO., LTD